

Power Stand Frame Maintenance Manual



- It is recommended that bungees are replaced every 250 uses.
- Check and tighten screws on bungee attachment knobs (monthly)
- Check caster and bottom plate screws monthly. NOTE: It is highly recommended that the casters are removed after install. They are for install and transportation purposes only.
- One seat assembly (optional) Remove 4 T-bolt screws from each seat. Center seat in middle of frame over bolt pattern. Align over holes and screw in T-bolts



Accessory Information

- Bungee



Safe working load limit – 150 pounds
Maximum load (break strength) – 300 pounds
Replacement price - \$28.50 + shipping

NOTE

The weight of the patient must not exceed the combined working load limit of the bungees being used. For example, if you are using bungees with a working load limit of 150 lbs each to support a patient weighing 300 lbs, then you need at least 2 bungees to safely secure that patient.

Order Form

	Quantity	Price	Total
Bungee	_____	\$28.50 ea	_____

Shipping at cost

Normal delivery: 10 – 14 days

Rush service available at additional charge

Send order to: operations@powerneurorecovery.com

Provide ship to address and contact:

Power Stand Frame Patient Use Instructions

